


# GRAFTING ASSEMBLY

Read instructions carefully before use

lipografter®

- 1 Remove all items from AT-Valve®. Switch cap to available port on AT-Valve® according to diagram
- 2 **2a** » Screw syringe onto AT-Valve®  
» open female Luer-Lock port  
**2b** » Connect cannula to the opposite side of AT-Valve®
- 3 Connect collection bag tubing onto remaining port
- 4 Open clip on bag to allow the adipose tissue to flow through device
- 5 Use small volume syringe to remove fat from collection bag and deposit it through the injection cannula

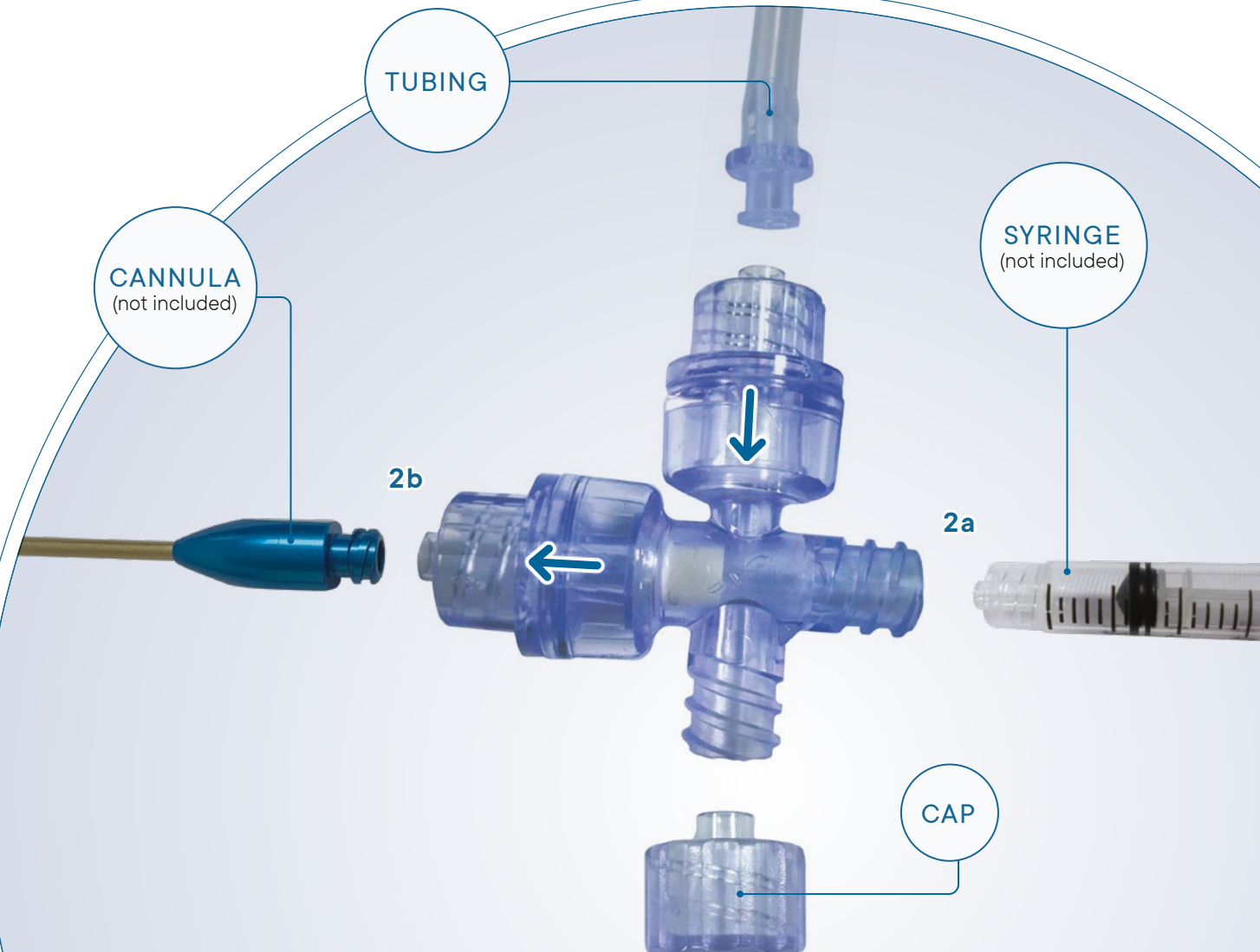

International Distributor  
**POLYTECH**

\* DO NOT OVER-TIGHTEN COMPONENTS

LipoGrafter®, KVAC®, and AT-Valve® are registered trademarks of Lipocosm LLC.

## HARVESTING ASSEMBLY

Read instructions carefully before use



### Materials Included in Kit:

- » KVAC® LipoGrafter® system (vacuum harvesting device)
- » Atraumatic tissue valve (AT-Valve® and cap)
- » LipoGrafter® Tube set and bags
  - › 4 Storage bags (250 ml each)
  - › Tubing for connection of components

### Other Materials Required

(not included in kit):

- » Cannula set (small diameter, multi-hole harvesting cannula) and injection cannulas with Luer-Lock fitting
- » Luer-Lock syringe
- » Sharp needle (large bore needle recommended)



- 1 Make sure clear cap is screwed onto AT-Valve®
- 2 Connect KVAC® to open female Luer-Lock port
- 3 Connect tubing directly to AT-Valve®
- 4 Attach storage bag directly to AT-Valve®
- 5 Connect cannula to opposite end of tubing

\* DO NOT OVER-TIGHTEN COMPONENTS

International Distributor  
**POLYTECH**

LipoGrafter®, KVAC®, and AT-Valve® are registered trademarks of Lipocosm LLC.